



Meets ISO 11140-1:2005!



Passed Examples



Failed Examples



STEAMPlus™ Steam Sterilization Integrator Instructions for Use:

1. For the highest level of sterility assurance, place a STEAMPlus Integrator in every pack.
2. Process the load according to the sterilizer manufacturer's instructions.
3. Before using items, observe the STEAMPlus Integrator for proper movement into the blue SAFE area.

Special Note:

Color of the bar is not an indication of a pass or fail result. If the color bar has not entered into the blue SAFE area, DO NOT use. Proper sterilization has NOT been achieved and the items in the package should be reprocessed.

For questions about your sterilization processes, please contact:

SPSmedical Supply Corp. • 6789 W. Henrietta Road • Rush, NY 14543 USA
 (585) 359-0130 • (800) 722-1529 • Fax: (585) 359-0167 • E-mail: info@spsmedical.com
 Visit our website: www.spsmedical.com